


# SD Ceraam

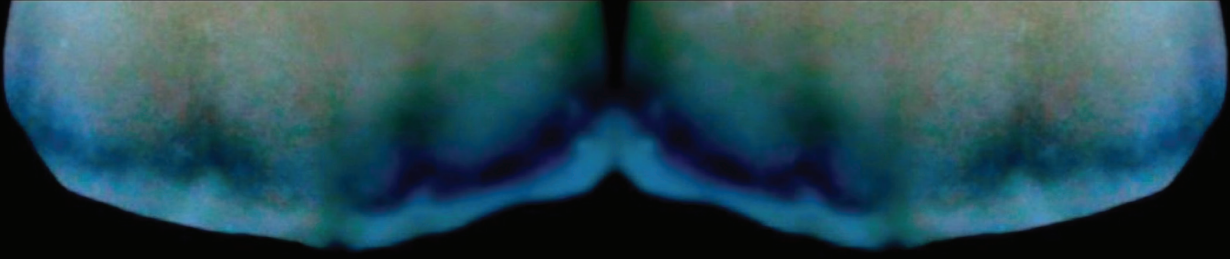



**ONE BAKED**

# SD Ceram

**Smart Dentin**


**Chroma**



**Mamelon**



**Modifier**

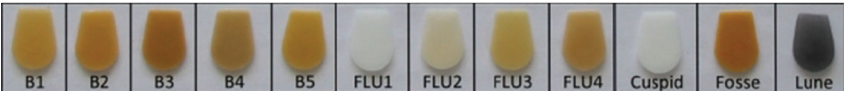

**Gingiva**



**Bleach Shade**




**Enamel**

**Enamel Intensive**



**Enamel Occusal**

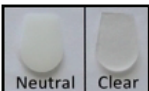

**Enamel Opal**



**Enamel**

**Translucent**




**Translucent**




# SD Ceram


Opaque Powder

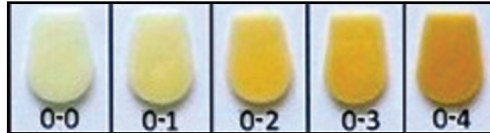

Opaque Dentin

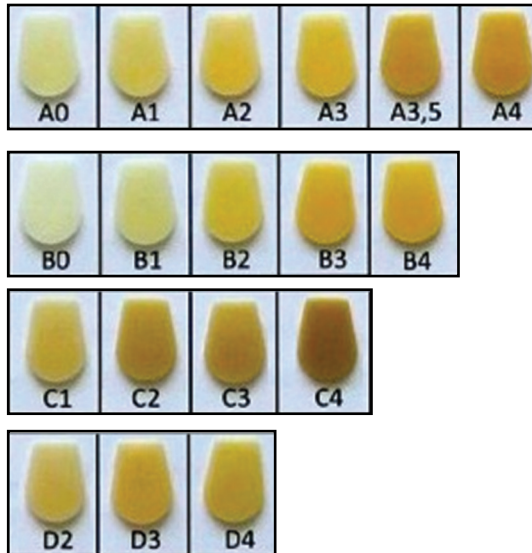

Smart Dentin

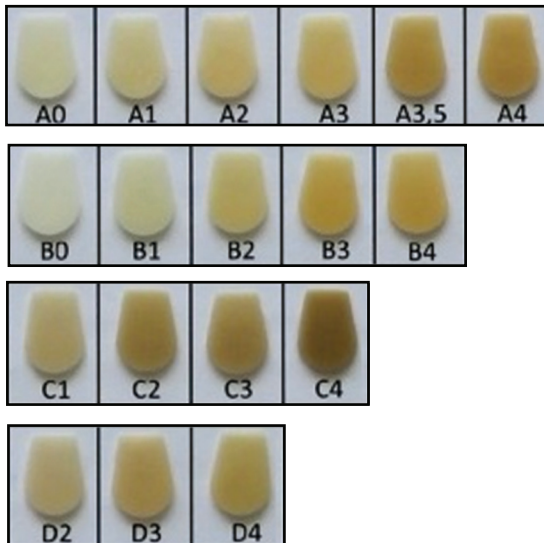

**SD** *Ceram*

ZR ceramic system

Firing instruction

	Preheating Temp	Drying Time	Vacuum Start	Heat Rate	Vacum Stop Final Temp	Holding Time	Cooling
<b>Pure Firing</b>	500°C	3 min	500°C	65°C / min	980	5 min	3 min
<b>1st Dentin Firing</b>	450°C	6 min	450°C	45°C / min	810°C	1 min	3 min
<b>2nd Dentin Firing</b>	450°C	6 min	450°C	45°C / min	810°C	1 min	3 min
<b>Glaze Firing</b>	450°C	3 min	-	45°C / min	800°C	1 min	3 min

# SIGMADENT®



**SIGMADENT®** by **Asfarma**  
DENTAL LABORATUVAR MALZEMELERİ, KLİNİK CİHAZLAR Medikal Dental Ürünler ve İlaç San. A.Ş.

Oğuzhan Cd. Akkoyunlu Sk. 7/B 34093  
Fındıkzade-Fatih İSTANBUL - TÜRKİYE  
Tel: +90 212 533 50 00  
Faks: +90 212 533 50 00  
Gsm: +90 532 247 66 14

**Skype:** sigmadental  
**FaceBook:** Sigmadent  
**Whatsapp:** +90 532 247 66 14  
**e-mail:** info@SigmaDent.com.tr  
**web:** www.SigmaDent.com.tr